424402A

Algoplaque Film-Thin

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EN EXTRA-THIN STERILE HYDROCOLLOID DRESSING

PRESENTATION

Algoplaque Film/Thin is a sterile adhesive dressing consisting of an hydrocolloidal suspension laid on an extra thin backing.

PROPRIÉTÉS

Algoplaque Film/Thin forms a liquid gel when in contact with the wound creating favourable conditions for the healing process.

CE 0459 (2002)

(2019-02)

STERILE R

Made in France

Laboratoires URGO - 42 rue de Longvic - 21300 CHENOVE - France

Algoplaque Film/Thin is translucent allowing the wound to be continually monitored.

Because it is flexible Algoplaque Film/Thin can be applied easily to all parts of the body and as it is Water proof and bacterial proof, baths and showers can be taken.

INDICATIONS

Algoplaque Film/Thin is a dressing designed for the local treatment of low exuding superficial wounds (ulcers, superficial pressure sores, injuries), almost-healed lesions (epithelising stage) and sutures.

In that case, the dressing should be changed every 4 to 6 days.

INSTRUCTIONS FOR USE

1. Preparation of the wound:

- Clean the wound normally and dry around the edges with a sterile compress.

- If the wound is contaminated, clean using an antiseptic, rinse with a saline solution and dry around the edges.

2. Application of the dressing:

Remove the protective tabs and apply Algoplaque Film/Thin without stretching it to the wound taking care not to touch the adhesive area with your fingers. Apply the dressing leaving a 2 to 3 cm margin covering healthy skin. Apply the dressing gently to the wound pressing more firmly around the edge.

3. Changing the dressing:

It is better to wait that the dressing was partly removed before changing it. Because a too early dressing changing could damage healthy skin.

Do not remove the dressing before 48 hours.

In order to ease the withdrawal, remove the dressing parallel to the skin.

On removal of the dressing the wound maybe covered in residue, this can easily be removed causing no pain by rinsing with saline or water.

WARNINGS AND PRECAUTIONS FOR USE

- Do not partly remove the dressing in order to monitor the wound because it could increase the risk of removal.

- The use of Algoplaque Film/Thin for ulcers of venous origin does not replace the need for compression treatment where prescribed.

- The use of Algoplaque Film/Thin on pressure sores does not replace the need for normal nursing care.

- Algoplaque Film/Thin must not be used in a hyperbaric chamber.

- Single use sterile individual packaging; re-using a single use dressing may lead to risks of infection.

- (*) Check that the sterile protector is intact before use. Do not use if package is damaged.

- Do not re-sterilise the dressing.

- Store Algoplaque Film/Thin dressings flat.

CONTRAINDICATIONS

- Known sensitivity to the dressing.

- Do not use Algoplaque Film/Thin for infected wounds.

- Do not use Algoplaque Film/Thin on severely exuding wounds.

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